

# Wash-Checks®

Disposable Washer-Disinfectant Monitor & Reusable Holder

## How do you know it's really clean?

- Use Wash Checks to monitor the efficacy of your cleaning process when using washer-disinfectors
- Be confident the cleaning was done properly – you cannot sterilize without proper cleaning
- Non toxic red test soil mimics the presence of blood and tissue on an instrument surface



### EASY TO INTERPRET



**Unexposed**  
The red spot will disappear when proper washer cycle conditions have been met



**Pass**  
Effective cleaning



**Insufficient**  
If any red remains, the test is insufficient. Proper washer cycle conditions have not been met


### Multi-Parameter Test

- Time
- Temperature
- Detergent Concentration
- Spray arm function
- Enzyme soak



Always refer to the manufacturer's instructions for use, national and local regulations and requirements of your regulatory agency.

### Insufficient Results

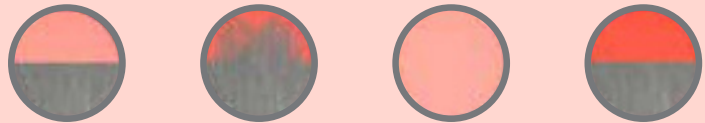

### Common Failures

- Overloading
- Inadequate water spray/impingement
- Clogged spray arms
- Inadequate/improper detergent dosing
- Poor water quality



Place a hash mark or circle representing the position of the Wash-Checks on the location rectangle that simulates an imaginary washer basket. This allows for placement tracking.



### Cleaning Monitor Holder

- Holder acts as a hinged instrument measuring impingement in washer-disinfectors
- Reusable stainless steel holder secures monitor in place in order to provide accurate results

Product	Code	Qty
Wash-Checks® Test Strips	WC-101	100/bag
Wash-Checks® Reusable Holder	WC-108-HOLDER	1

1.800.265.9931  
www.germiphene.com

