



Wash-Checks[®]
Cleaning Monitor
 REF WC 101

Directions For Use

INDICATIONS FOR USE

The Wash-Checks Disposable Cleaning Monitor is designed for monitoring cleaning efficiency during the wash process.

INSTRUCTIONS FOR USE

1. Insert a Wash-Checks monitor (WC 101) into a Wash-Checks Holder (WC 102) completely. The front side of the monitor with red test soil will be visible in the holder window.
2. Position Wash-Checks Holder face up inside a basket. Place basket on a shelf in the washer. Vary positions and shelves of Wash-Checks placement cycle to cycle.
3. Run a complete washer cycle.
We recommend monitoring every shelf on every washer each day for a machine release and using a minimum of one WashChecks per load for routine monitoring throughout the day, or as per manufacturer's instructions.
4. At the end of the washing cycle, remove the Wash-Checks monitor from the Holder and read the result
Caution: Wash-Checks Holder and monitor strip may still be hot! Remove with care.
5. Record results.

INTERPRETATION OF RESULTS



Untreated monitor, or critical/poor result (entire red circle is visible)



Insufficient result (some red remaining in circle)



Optimum result (no red remaining in circle)

You may also record the location of monitor placement in the washer on the Wash-Checks with a fine tip waterproof marker.



Place a hash mark or circle representing the position of the Wash-Checks on the location rectangle that simulates an imaginary washer basket. This allows for placement tracking.

STORAGE AND HANDLING

- Store Wash-Checks monitors inside the protective foil bag provided at 40°–86°F (4°–30°C).
- Do not remove Wash-Checks monitors from foil bag until ready for processing.
- Keep away from disinfectants and fumes.

EXPIRY DATE

The expiry date is printed on the product packaging.



TO REORDER

1.800.265.9931

www.germiphene.com