5% Sodium Fluoride Varnish

Product Description

D-Lish Sodium Fluoride-containing Desensitizing Varnish

is a fluoride containing dental varnish for surface application to enamel and dentin. D-Lish Sodium Fluoride-containing Desensitizing Varnish will adhere to wet surfaces and is tolerant to moisture and saliva.

D-Lish Sodium Fluoride-containing Desensitizing Varnish is an ethanolic suspension of colophony with artificial flavors and sweetened with xylitol.

1 ml of D-Lish Sodium Fluoride-containing Desensitizing Varnish contains 50 mg of sodium fluoride equivalent to 22.6 mg of fluoride.

Indications:

D-Lish Sodium Fluoride-containing Desensitizing Varnish is intended for use as:

- Treatment of hypersensitive teeth
- Treatment of sensitive root surfaces

Contraindications:

D-Lish Sodium Fluoride-containing Desensitizing Varnish is contraindicated in patients with ulcerative gingivitis or stomatitis or known allergies to colophony.

D-Lish Sodium Fluoride-containing Desensitizing Varnish is not intended for systemic treatment.

Methods of Use:

Recommended dosage:

- Treatment of patients with primary dentition. Use approx. 0.25 ml of D-Lish Sodium Fluoride-containing Desensitizing Varnish
- Treatment of patients with adolescent or permanent dentition. Use approx. 0.40 ml of D-Lish Sodium Fluoride-containing Desensitizing Varnish

D-Lish Sodium Fluoride-containing Desensitizing Varnish should be applied on a complete set of teeth in a very thin film thickness. A thick film thickness is more likely to peel off and does not provide better therapeutic results. A complete set of teeth can be treated between the ranges of 0.25ml and 0.40ml.

Application:

1. Thoroughly clean tartar from treatment areas.
2. For best results, eliminate any excessive moisture/saliva from area to be treated.
3. Use brush applicator or foam pellet to uniformly apply D-Lish Sodium Fluoride-containing Desensitizing Varnish, covering the entire surface to be treated as a thin film.
4. Allow area to become wet (either by gentle rinsing or natural salivary flow) to ensure setting of D-Lish Sodium Fluoride-containing Desensitizing Varnish.
5. Instruct patient to avoid hard food, alcohol, brushing or flossing for the next 4 hours after application.

Precautionary measures:

Edematous swellings have been reported on rare occasions especially following application to extensive surfaces. Direct ingestion should be avoided. Dyspnea associated with asthmatic children, has been reported for similar preparations in extremely rare cases. Patients having sensitive digestive systems may experience nausea, usually after extensive treatment. In case of adverse reactions, remove varnish with a soft toothbrush and floss, and then rinse with warm water.

No treatment with other fluoride preparations on day of application. Suspend treatment with fluoride tablets for several days after treatment. Do not swallow.

According to its purpose, D-Lish Fluoride-containing Desensitizing Varnish has a sticky consistency. Material remaining on the screw thread may cause the cap to stick.

Wipe off any contaminating residues with an ethanol containing cloth.

Storage: Store at temperatures between 39°F - 73°F (4°C - 23°C). Do not use after expiration date.

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. D-Lish recognizes its responsibility to replace products if proven to be defective. D-Lish does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

Keep this material out of reach of children.

For dental use only.

For emergencies contact INFOTRAC at 1-800-535-5053.

Single Use
Only

**Rx
ONLY**

YOUNG
INNOVATIONS

Manufactured For:
Young Dental Manufacturing Co.
13705 Shoreline Court East
Earth City, MO 63045 USA
800-325-1881 • www.younginnovations.com
Made in Germany

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