

YOUNG® PerioDT®

WITH DESICCATION TECHNOLOGY

Application Process

STEP
01

ISOLATE & DRY

After identifying appropriate patient for PerioDT treatment, isolate the surface where PerioDT will be placed. Dry the surface with air or gauze.



STEP
02

APPLY

Apply PerioDT subgingivally until product flows from the pocket, leaving a thin layer supragingivally. Do not overfill. Leave the product on the surface for no longer than 10-30 seconds.



STEP
03

SUCTION & RINSE

First, start by using a HVE to suction off any remaining product from the surface. Then, thoroughly rinse the mouth out with water. Do not leave any residual product in the patients mouth.



STEP
04

INSTRUMENTATION

Perform scaling and root planing treatment.



STEP
05

REAPPLY

As a final step, repeat PerioDT application steps 1-3 to destroy any remaining biofilm.



NOTE: When utilizing PerioDT for periodontal treatment and/or heavy debridement, it is recommended to thoroughly evacuate and rinse all PerioDT before beginning your scaling procedure. Then, if needed, reapply to periodontally diseased pockets and follow the instructions for use. **Additional adjunct treatments are not necessary or recommended following PerioDT application. Additional medicaments placed in periodontal pockets may increase the risk of post treatment irritation or inflammation. Patients may resume normal home care routines and there are no dietary restrictions immediately post treatment.**

Indication and Workflow for Periodontal Treatment

Identify appropriate
patient for PerioDT
treatment



Isolate and Dry the Surface



Start by applying PerioDT
subgingivally (if applicable)
and move supragingivally



Suction and Rinse



Scaling and Root Planing



Repeat the above
application steps to
apply post SRP

Reference Instructions for Use for additional application tips and safety information at bit.ly/perioddt-ifu